



Revolutionizing the acne routine with no routine at all

Get to know the first and only FDA-cleared energy device for the treatment of mild, moderate and severe acne. AviClear™ delivers lasting results in three 30-minute treatment sessions – no downtime, no prescriptions and no need for your old acne routine.^{1,2}



Understanding acne



Model. Not actual patient.

Ready for a future without worrying about your acne?

You're not alone.

85%

of teens have acne.³ For many, it can persist into adulthood.³

But that doesn't mean you have to live with it.


Let's change how you look at your acne

You've probably heard all sorts of reasons why people get acne. Most are myths, and the truth may surprise you. It really isn't your fault. **Here are the facts**^{4,5}:

- Acne is **not a short-term condition**
- It is not caused by **dirty skin**
- **Diet** has little to no effect on your acne
- **Scrubbing** your skin many times a day will not get rid of acne
- **Genetics** can play a role in whether you develop acne and how severe it is

What really causes acne?

When your body produces excess sebum, the oils on your skin, it mixes with your dead skin cells and clogs your pores. Bacteria often gets trapped in the clogged pores and triggers inflammation, causing red and pus-filled pimples.⁶

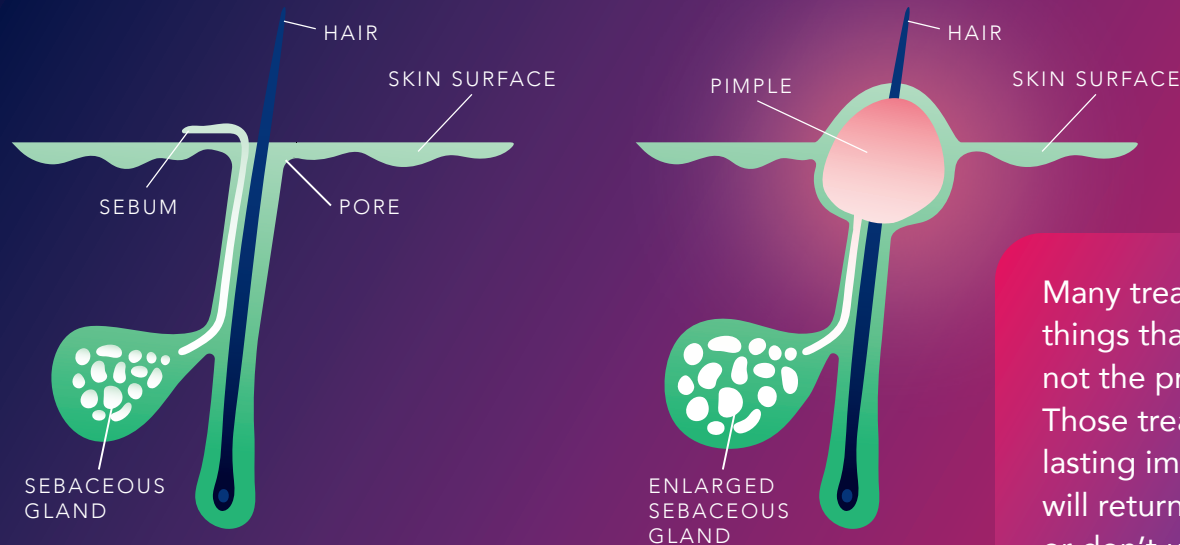

Many treatments target other things that cause acne, but not the production of sebum.⁷ Those treatments won't provide lasting improvement. Your acne will return if you stop treatment or don't use it every day.^{8,9}

How severe is your acne?^{2,10}



MILD

Few inflammatory lesions

No nodular lesions



MODERATE

Multiple inflammatory lesions

No more than 2 nodules



SEVERE

More apparent inflammatory lesions

No more than a few nodules

The more severe your acne, the more challenging it is to treat.



What is AviClear?



Model. Not actual patient.

What is AviClear?

This is the future of acne treatment

AviClear is the only laser treatment that delivers long-lasting results for all severities of acne in all skin types.

ADVANCED

First and only FDA-cleared energy device for the treatment of mild, moderate and severe acne

FAST

Lasting clearance in only three 30-minute treatment sessions – with no downtime and no need for adjunctive pain therapy^{1,2}

SAFE

Drug-free, nontoxic and no harmful side effects²





AviClear™

CUTERA

AviClear works differently

AviClear uses exclusive technology to specifically target the source of sebum, the oils on your skin. After treatment, you will produce less oil and your acne will get better.*

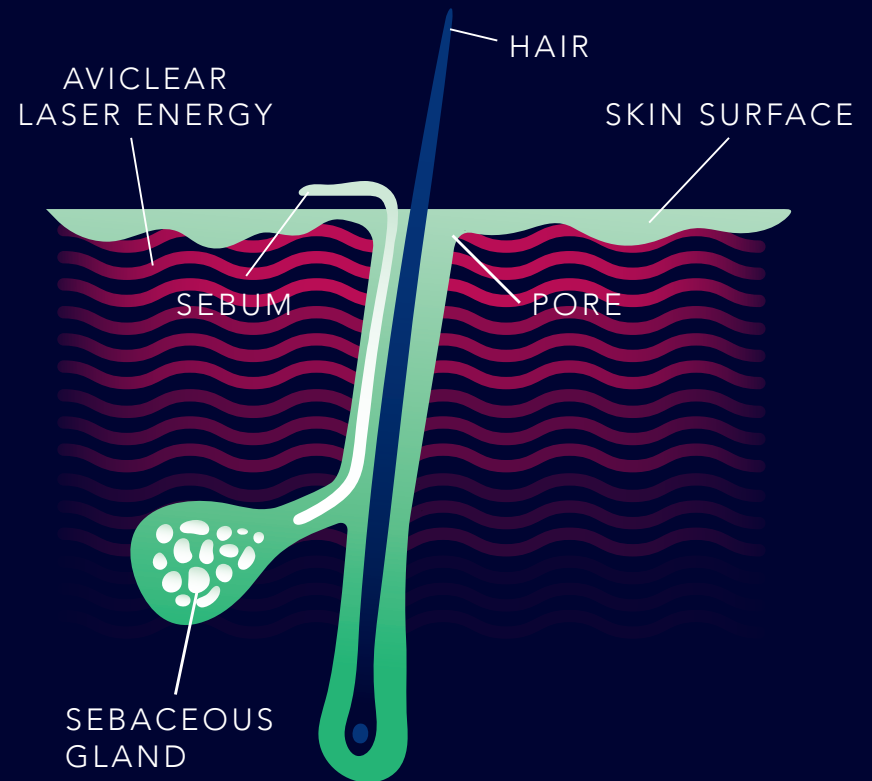

*In clinical studies, 80% saw at least half of their acne go away three months after their final treatment session and 87% achieved this by six months.²

The difference is clear

Mild Moderate Severe

AviClear™



- FDA-cleared to treat all severities of acne
- Just three quick 30-minute sessions
- Long-lasting results¹
- Drug- and chemical-free
- No harmful side effects or downtime²
- Safe for all skin types

Benzoyl peroxide/ topical retinoids¹¹⁻¹⁴



- FDA-approved to treat all severities of acne
- Applied daily until acne improves on its own, which could be several years
- Must be used continuously to maintain acne control

Photodynamic therapy¹⁵⁻¹⁷



- Not FDA-cleared to treat acne*
- Three to four 2.5-hour sessions
- Results are semi-permanent, meaning more treatments may be required
- Pain has been reported to be severe

Oral antibiotics¹⁸



- FDA-approved, usually used in moderate to severe acne
- Taken daily; used for the shortest time possible to prevent antibiotic resistance
- May require multiple courses
- May take several months or years for acne to improve

Isotretinoin¹⁹



- FDA-approved to treat severe nodular acne that does not respond to other treatments
- Daily doses taken over 15-20 weeks per course
- Boxed warning, with many restrictions and serious side effects
- Serious side effects, including birth defects, mental health problems, brain and skeletal problems
- Suicide warning
- Required participation in the risk evaluation and mitigation strategy, iPLEDGE

*Blu-U(R), one component of one of photodynamic therapy system has been FDA-cleared to treat moderate acne.

See your skin in a whole new light


In clinical studies:

- 87% of patients saw at least half of their acne clear in the first six months after treatment (up from 80% at three months) ²
- Results may continue to improve with time, even several months after your treatment sessions are finished¹

87%



**See the
results**

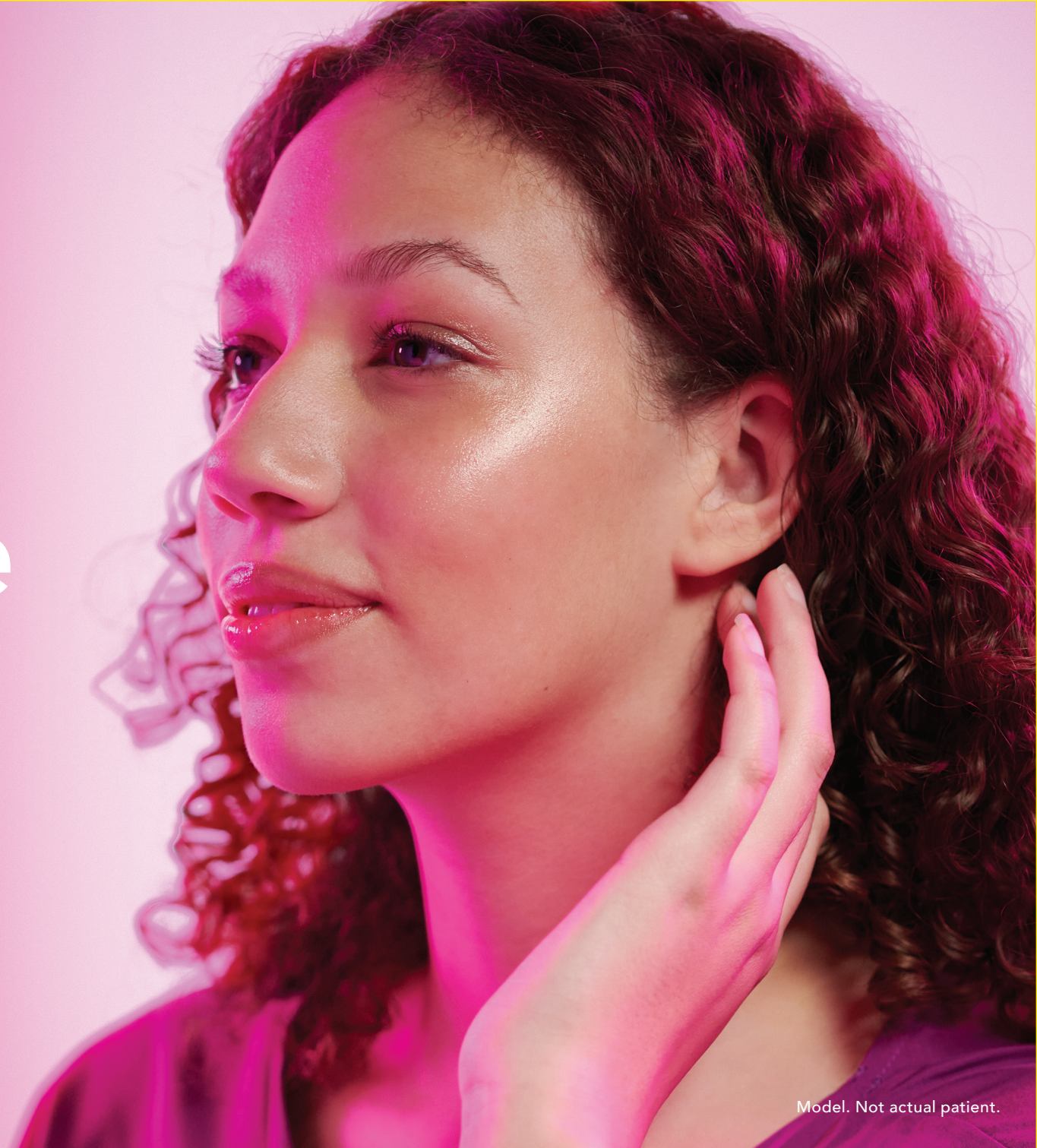

Model. Not actual patient.

[See the results](#)

The future is so much clearer now

Results that redefine what's possible without a prescription²

Moderate to clear

AGE: 17

SKIN TYPE: IV

Moderate | Baseline



Clear | 3 months after final treatment session



Clear | 6 months after final treatment session



The future is so much clearer now

Results that redefine what's possible without a prescription²

Severe to moderate

AGE: 26

SKIN TYPE: VI

Severe | Baseline



Moderate | 3 months after final treatment session



Moderate | 6 months after final treatment session



The future is so much clearer now

Results that redefine what's possible without a prescription²

Moderate to almost clear

AGE: 17

SKIN TYPE: V

Moderate | Baseline



Almost Clear | 3 months after final treatment session



Almost clear | 6 months after final treatment session



The future is so much clearer now

Results that redefine what's possible without a prescription²

Moderate to clear

AGE: 21

SKIN TYPE: II


Moderate | Baseline



Almsot clear | 3 months after final treatment session



Clear | 6 months after final treatment session

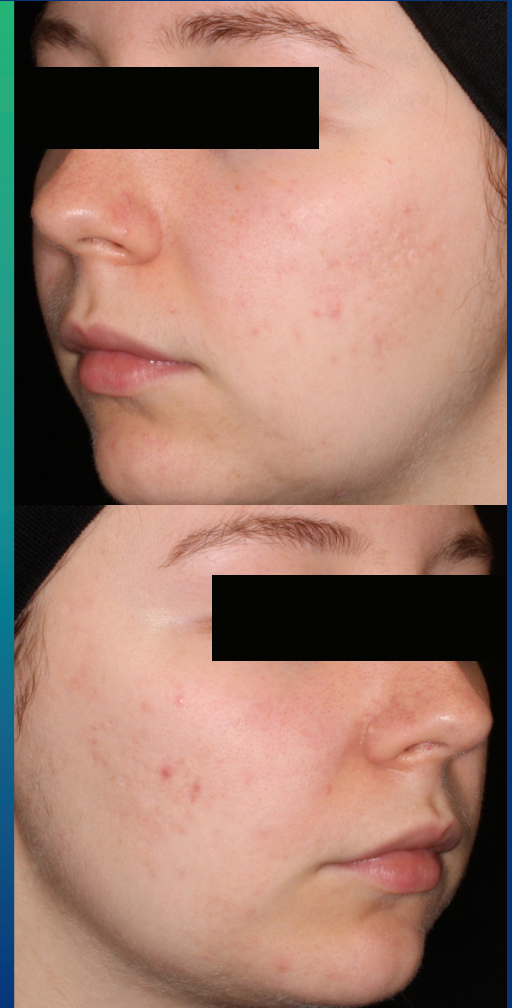

The future is so much clearer now

Results that redefine what's possible without a prescription²

Moderate to almost clear

AGE: 18

SKIN TYPE: IV

Moderate | Baseline



Almost clear | 3 months after final treatment session



Almost clear | 6 months after final treatment session



The future is so much clearer now

Results that redefine what's possible without a prescription²

Severe to mild

AGE: 25

SKIN TYPE: II

Severe | Baseline



Mild | 3 months after final treatment session



Mild | 6 months after final treatment session



Two years into a clearer future

Long-term results that redefine what's possible without a prescription¹

Moderate to almost clear
Two years post-treatment

AGE: 25

SKIN TYPE: III


Moderate | Baseline



Mild | 6 months after final treatment session



Almost clear | 2 years after final treatment session

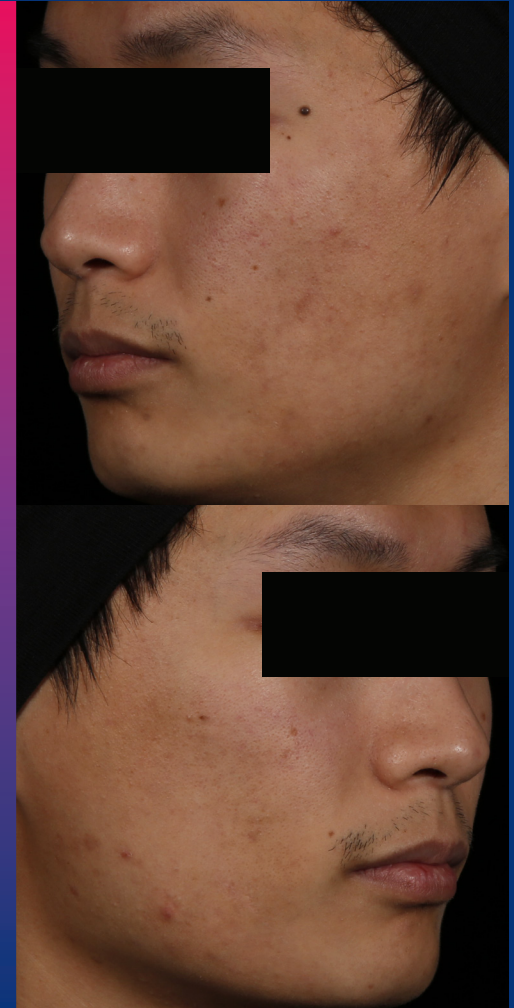

High patient satisfaction

87%

of patients were "satisfied" or "very satisfied" with their results¹

75%

of patients were "likely" or "very likely" to have the laser treatment again²



What to expect

Model. Not actual patient.

What to expect

What to expect on treatment days

If you and your provider decide that AviClear is right for you, treatment sessions will go something like this. **Your provider will:**

- Have you shave any excess hair
- Clean your skin
- Capture before treatment photos
- Perform your AviClear treatment session, which lasts around 30 minutes
- Apply a broad-spectrum SPF 30+ sunblock immediately after treatment
- Review your post-treatment care instructions and recommendations
- Schedule your remaining treatment session appointments

Twelve weeks after your final treatment, you will be asked to return for a final appointment to take your after photos and have your final assessment.

Making you comfortable during treatment

AviClear was designed with advanced technology, because the treatment experience matters.



Exclusive AviCool™ sapphire skin cooling and sensory controls for increased comfort



You may notice a light snapping sensation on your skin



Everyone in the clinical trials tolerated treatment well, without the need for adjunctive pain therapy or the need to stop treatment²



What to expect after your treatment



No serious **side effects** from treatment²



You may experience **mild redness or swelling** that could last from one hour up to two days²



No **downtime**²



You may also experience **acne flareups**, but they are temporary and should go away with time²



You should apply SPF 30+ sunblock daily and **avoid direct sunlight** throughout your treatment duration



A clear future



Model. Not actual patient.

A clear future

Is AviClear right for you?

You may be an ideal AviClear patient if you are:



Adolescent or older



Searching for a safe, effective and drug-free alternative to traditional acne treatments



Suffering from mild, moderate or severe acne




Wanting your acne treatment to be complete in a few months instead of lasting many years


AviClear is safe to use for all severities of acne in all skin types!

You may experience redness, swelling, acne flareups, dryness or itchiness but these should be mild, transient and self-resolving.²

It's time to put your old acne routine in the past

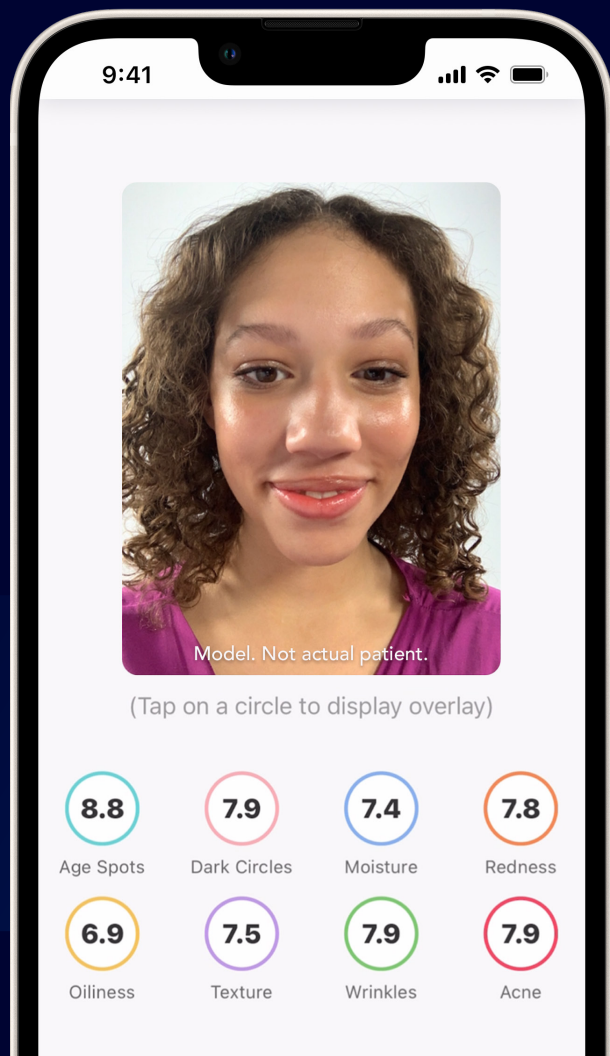

A quick personalized, digital skin analysis* with the Cutera App is a great first step. It only takes a few seconds to see where your skin health stands.

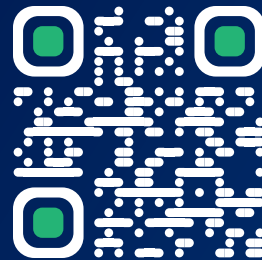

Just scan the QR code to download the app and get started.

*Skin analysis not available on Android.



References

1. Data on file. Cutera Inc.
2. Data on file. FDA clearance study. Cutera Inc.
3. Bhate K, Williams HC. Epidemiology of acne vulgaris. *Br J Dermatol*. 2013;168(3):474-485.
4. Mayo Clinic. Acne. Available from <https://www.mayoclinic.org/diseases-conditions/acne/symptoms-causes/syc-20368047>. Accessed Feb. 24, 2022.
5. National Institute for Health Research. Misconceptions about acne lead to underuse of effective treatments; people need reliable information to manage the condition long-term. Available from <https://evidence.nihr.ac.uk/alert/misconceptions-acne-lead-to-underuse-effective-treatments-reliable-information-needed/>. Accessed Feb. 24, 2022.
6. Makrantonaki E, et al. An update on the role of the sebaceous gland in the pathogenesis of acne. *Dermatoendocrinol*. 2011;3(1):41-49.
7. Chovatiya R. Acne treatment. *JAMA*. 2021;326(20):2087.
8. Balkrishnan R, et al. Predictors of healthcare outcomes and costs related to medication use in patients with acne in the United States. *Cutis*. 2006;77(4):251-255.
9. Tan AU, et al. A review of diagnosis and treatment of acne in adult female patients. *Int J Womens Dermatol*. 2018;4(2):56-71.
10. Gold MH, et al. Self-diagnosis of Mild-to-Moderate Acne for Self Treatment with Blue Light Therapy. *J Clin Aesthet Dermatol*. 2009;2(4):40-44.
11. Thielitz A, et al. Topical retinoids in acne vulgaris: update on efficacy and safety. *Am J Clin Dermatol*. 2008;9(6):369-381.
12. Drugs.com. Tretinoin topical. Available from <https://www.drugs.com/mtm/tretinoin-topical.html>. Accessed Feb. 28, 2022.
13. Palmer A. Why pimples can return after acne treatment. Available from <https://www.verywellhealth.com/pimples-come-back-when-i-stop-using-acne-medicine-15528>. Accessed Feb. 28, 2022.
14. NHS. Benzoyl peroxide. Available from <https://www.nhs.uk/medicines/benzoyl-peroxide/>. Accessed Feb. 28, 2022.
15. Durango Dermatology. Levulan Photodynamic Therapy. Available from <https://www.durangodermatology.com/storage/app/media/levulan20photodynamic20therapy20updated.pdf>. Accessed Feb. 24, 2022.
16. Warren CB, et al. Pain Associated with Aminolevulinic Acid-Photodynamic Therapy of Skin Disease. *J Am Acad Dermatol*. 2009;61(6):1033-1043.
17. Leonard J. The benefits and side effects of blue light treatment for acne. Available from <https://www.medicalnewstoday.com/articles/319256>. Accessed Mar. 2, 2022.
18. Mayo Clinic. Acne diagnosis & treatment. Available from <https://www.mayoclinic.org/diseases-conditions/acne/diagnosis-treatment/drc-20368048>. Accessed Feb. 28, 2022.
19. Isotretinoin FDA Prescribing Information. Available from <https://www.drugs.com/pro/isotretinoin-capsules.html>. Accessed Feb. 24, 2022.